

For further assistance, please
visit [Tenovi.com/support](https://tenovi.com/support).

tenovi

© 2025 Tenovi Co.
All rights reserved. Republication, duplication or modification
in whole or in part is prohibited without prior written permission
from Tenovi. All trademarks are owned by or licensed to
Tenovi, Co. unless otherwise noted.
Revision 05.01.2024

1

Power Your Gateway


Plug your Gateway into a power outlet with the provided power cord and wait until the LED on the Gateway turns **RED**.



2

Insert Batteries


Insert the 2 AAA batteries that are included.

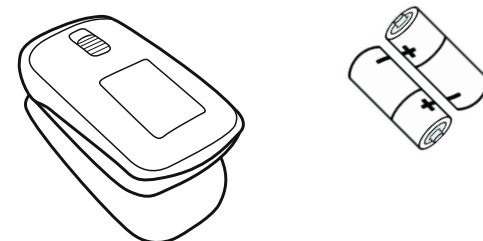

Insert Finger Into Pulse Ox

3


Insert **one finger** completely into the measurement section of the Oximeter with your **finger nail pointing up** towards the screen.

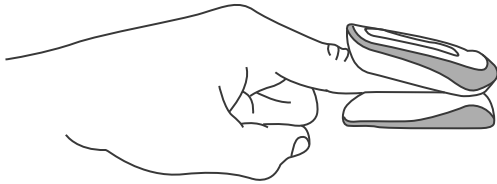

Wait 20 seconds

5


Keep your finger in the oximeter for **20 seconds** or until you hear a beep from the Tenovi Gateway. Then, **remove your finger** and wait for Tenovi Gateway to turn **GREEN**.

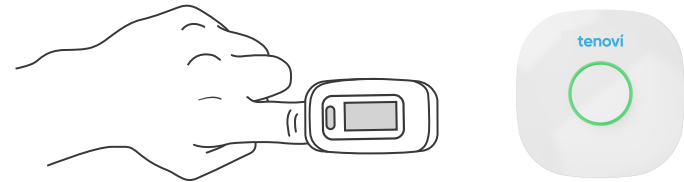

Press Power On Pulse OX

4

Stand within **20 feet** of the Tenovi Gateway and **press the power button** on the oximeter.

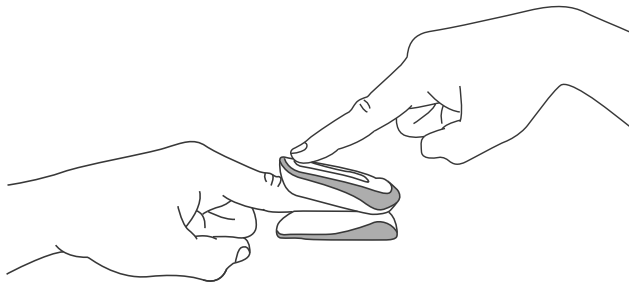

Daily Measurements

6

Take **one measurement** each day unless advised otherwise by your healthcare provider. The LED will turn **RED** each morning and then turn **GREEN** once a measurement has been successfully transmitted.



Measurement Needed



Measurement Successful