

For further assistance, please
visit [Tenovi.com/support](https://tenovi.com/support).

tenovi

© 2025 Tenovi Co.
All rights reserved. Reproduction, duplication or modification
in whole or in part is prohibited without prior written permission
from Tenovi. All trademarks are owned by or licensed to
Tenovi, Co. unless otherwise noted.
Revision 05.01.2024

1

Power Your Gateway


Plug your Gateway into a power outlet with the provided power cord and wait until the LED on the Gateway turns **RED**.



2

Remove battery cover


Remove the battery cover on the back of the scale, and insert the **4 AAA** batteries.

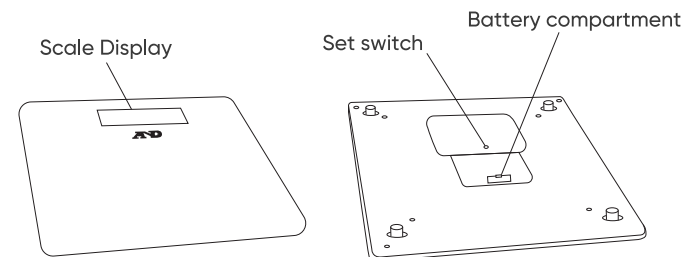

Scale Placement

3

Place the scale on a hard, even surface.
Avoid using the device on carpet.

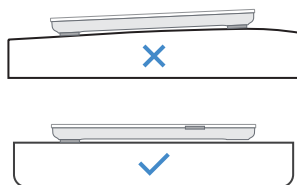

- A) Place your scale on a flat and hard surface.
- B) Do NOT place the scale on carpet or soft surface.

Take a measurement

5


When the measurement concludes, the Gateway will beep and the LED ring will turn **green**, indicating that you have properly taken a measurement, and that measurement was successfully transmitted.



Properly set up

4

Step on the scale and remain standing until the screen displays "**step Off.**" Your weight will be displayed once you step off the scale.

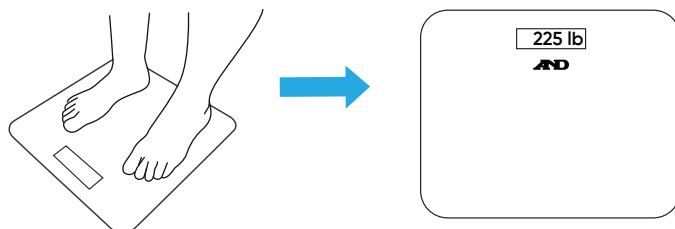

Daily Measurements

6

Take one measurement each day unless advised otherwise by your healthcare provider. The LED will turn **RED** each morning and then turn **GREEN** once a measurement has been successfully transmitted.



Measurement Needed



Measurement Successful