

1

Power Your Gateway

Plug your Gateway into a power outlet with the provided power cord and wait until the LED on the Gateway turns **RED**.



For further assistance, please visit [Tenovi.com/support](https://www.tenovi.com/support).

tenovi


© 2025 Tenovi Co.
All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Tenovi. All trademarks are owned by or licensed to Tenovi, Co. unless otherwise noted.
Revision 05.01.2024

2

Attach to inhaler

Place the Ellipta[™] into the Hailie[®] sensor. Gently push and slide the Ellipta[™] down until it clicks into position.


Place the Ellipta[™] into the Hailie[®] sensor

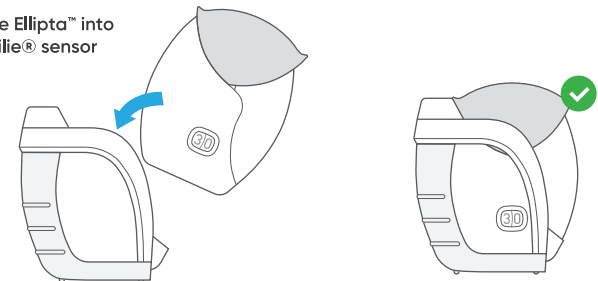

Bond to Gateway

3


Press the button three times to bond with the Gateway.

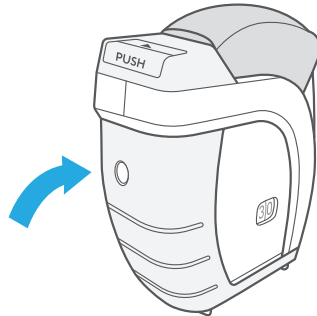

Delivering medication

4

Refer to the labelling provided with the inhaler for instructions on use. Carry out all steps required to use your inhaler according to the instructions.

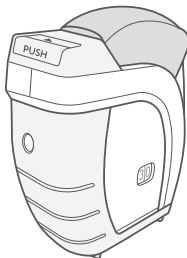

Daily measurements

5

When the measurement concludes, the Gateway will beep and the LED ring will turn **green**, indicating that you have properly taken a measurement, and that measurement was successfully transmitted.



For further assistance, please visit [Tenovi.com/support](https://www.tenovi.com/support).

tenoví