

1

Power Your Gateway

Plug your Gateway into a power outlet with the provided power cord and wait until the LED on the Gateway turns **RED**.



For further assistance, please visit [Tenovi.com/support](https://www.tenovi.com/support).


tenovi

© 2025 Tenovi Co.
All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Tenovi. All trademarks are owned by or licensed to Tenovi, Co. unless otherwise noted.
Revision 05.01.2024

2


Insert Batteries

Slide off back cover using your thumbs and insert two AAA batteries into the back of your pulse oximeter.

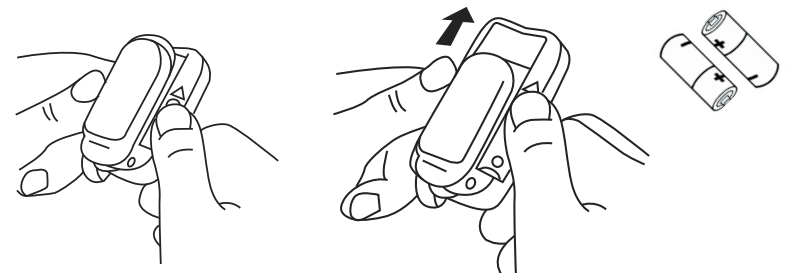


3 Insert Your Finger

Insert one finger completely into the measurement section of the pulse oximeter with your **finger nail pointing down, away from the screen**.

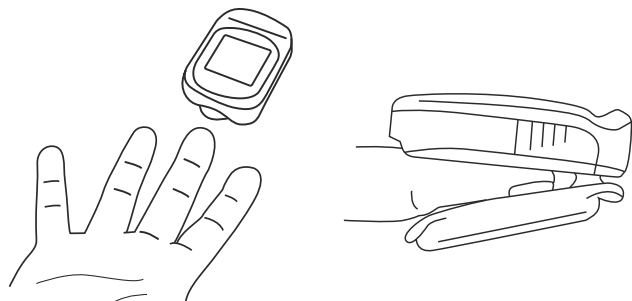

5 Wait For Results

Keep your finger in the Oximeter for **20 seconds** or until you hear a beep from the Tenovi Gateway. Then, remove your finger and wait for Tenovi Gateway to turn green.

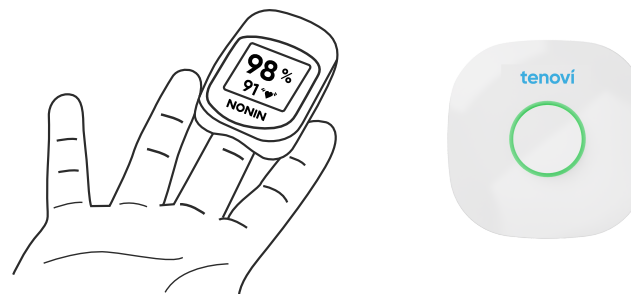

4 Take Your Measurement

Stand within 20 feet of the Tenovi Gateway. The pulse oximeter will automatically turn on and begin taking a measurement.



6 Daily Measurements

Take one measurement each day unless advised otherwise by your healthcare provider. The LED will turn **RED** each morning and then turn **GREEN** once a measurement has been successfully transmitted.



Measurement Needed



Measurement Successful