



For more information about your device, use your phone to scan the QR code above or visit our support site at [tenovi.com/07](https://tenovi.com/07)

tenovi

© 2026 Tenovi Co.

All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Tenovi. All trademarks are owned by or licensed to Tenovi, Co. unless otherwise noted.

Revision 06.11.26


## 1 Power Your Gateway

Plug your Gateway into a power outlet with the provided power cord and wait until the LED on the Gateway turns **RED**. If your Gateway is already plugged in, unplug and then plug it back in.



## 2 Remove your pill bottle cap

Once removed from packaging, pull the yellow tab to power on your PatchCap.

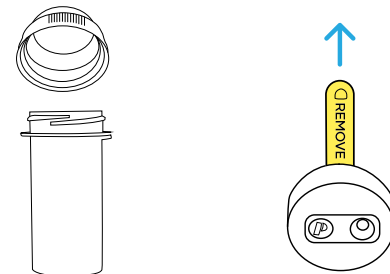

## Label your pill bottle cap

3


Use the colored sticker (red, blue, etc.) as a marker for PatchCap bottles by **peeling it off and applying it to the top of the bottle cap.**

Colored sticker

Transparent film






Patchcap

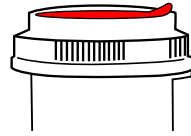Medication  
Pill Bottle

## Reattach the pill bottle cap



5

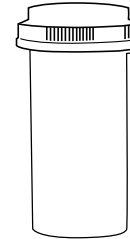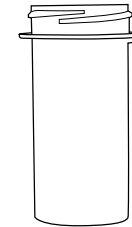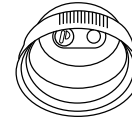

## Remove film and apply Patch

4

Next, **remove transparent film** on PatchCap to reveal adhesive. **Apply** the PatchCap to the inside of your bottle cap.

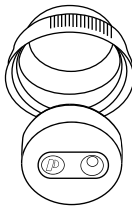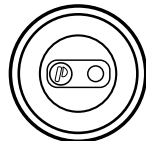Apply to inside  
of bottle cap

Bottom View

## Daily Measurements

6

Use your pill bottle like normal and take your medication as instructed by your healthcare provider. Your PatchCap will track when you use your medication. When in range of the Gateway, all open and close events will be transmitted to your care provider.



Measurement Needed



Measurement Successful