


# DEVICE SETUP GUIDE

## Part A PatchCap Setup


Let's begin with setting up your PatchCap.

Follow the steps below to set up your PatchCap prior to setting up your Gateway.

- 1**

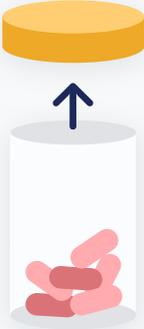


Remove your pill bottle's cap
- 2**

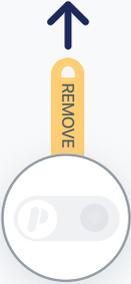

Remove the pull tab from the PatchCap
- 3**




Remove transparent peelable film to reveal adhesive backing
- 4**

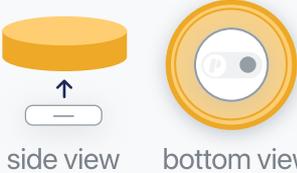


side view      bottom view

Apply the PatchCap to the inside of your bottle cap
- 5**

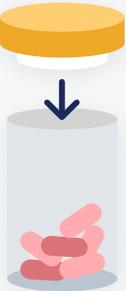

Reattach your pill bottle's cap
- 6**

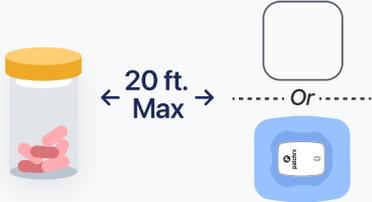

20 ft. Max      Or

Store medication in the same room as your Gateway, and take as prescribed