

For further assistance, please
visit [Tenovi.com/support](https://tenovi.com/support).

tenovi

© 2025 Tenovi Co.
All rights reserved. Republication, duplication or modification
in whole or in part is prohibited without prior written permission
from Tenovi. All trademarks are owned by or licensed to
Tenovi, Co. unless otherwise noted.
Revision 05.01.2025

Power Your Gateway

1


Plug your Gateway into a power outlet with the provided power cord and wait until the LED on the Gateway turns **RED**. If your Gateway is already plugged in, unplug and then plug it back in.



Remove your pill bottle cap

2

Once removed from packaging, **pull the yellow tab** to power on your PatchCap.

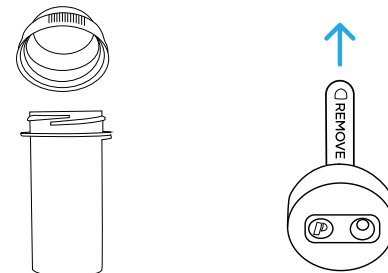


3 Mark your pill bottle cap

Use the colored sticker (yellow, blue, etc.) as a marker for PatchCap bottles by **peeling it off and applying it to the top of the bottle cap.**

Colored sticker Transparent film




Patchcap

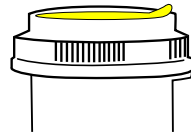


Medication
Pill Bottle

5 Reattach the pill bottle cap

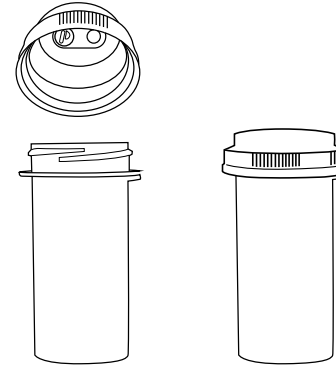


4 Remove film and apply Patch

Next, **remove transparent film** on PatchCap to reveal adhesive. **Apply** the PatchCap to the inside of your bottle cap

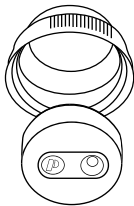

Apply to inside
of bottle cap

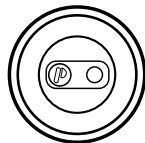

Bottom View

6 Daily Measurements

Use your pill bottle like normal and take your medication as instructed by your healthcare provider.

When in range of the Gateway, the PatchCap will connect and transmit open and close events to your care provider. Your gateway will turn from **red** to **green** if a medication event was sent that day.

Gateway connected

