For further assistance, please visit Tenovi.com/support.

tenoví

© 2025 Tenovi Co.

All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Tenovi. All trademarks are owned by or licensed to Tenovi, Co. unless otherwise noted.

Revision 10.23,2025


Power Your Gateway

Plug your Gateway into a power outlet with the provided power cord and wait until the LED on the Gateway turns RED. If your Gateway is already plugged in, unplug and then plug it back in.



Remove your pill bottle cap

Once removed from packaging, **pull the yellow tab** to power on your PatchCap.

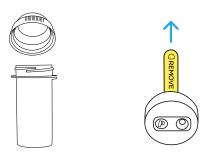

9

Label your pill bottle cap

Use the colored sticker (red, blue, etc.) as a marker for PatchCap bottles by **peeling it off and applying it to the top of the bottle cap.**

Colored sticker

Transparent film





Medication Pill Bottle


4

Reattach the pill bottle cap



Remove film and apply Patch

Next, **remove transparent film** on PatchCap to reveal adhesive. **Apply** the PatchCap to the inside of your bottle cap

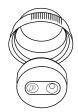



Apply to inside of bottle cap

Bottom View

Daily Measurements

Use your pill bottle like normal and take your medication as instructed by your healthcare provider.

Your PatchCap will track when you use your medication. When in range of the Gateway, all open and close events will be transmitted to your care provider.





Measurement Needed

Measurement Successful

6