

# DEVICE SETUP GUIDE

## Part A

## PatchCap Setup

Let's begin with setting up your PatchCap.

Follow the steps below to set up your PatchCap prior to setting up your Gateway.

- 

**1** Remove your pill bottle's cap
- 

**2** Remove the pull tab from the PatchCap
- 

**3** Remove transparent peelable film to reveal adhesive backing
- 

**4** Apply the PatchCap to the inside of your bottle cap
- 

**5** Reattach your pill bottle's cap
- 

**6** Store medication in the same room as your Gateway, and take as prescribed

# DEVICE SETUP GUIDE


## Part B

## Gateway Setup


### Now let's get your Gateway connected.

After setting up your PatchCap, follow the steps below to set up your gateway.

Once your PatchCap is setup, and your Gateway connected, you're ready to go!

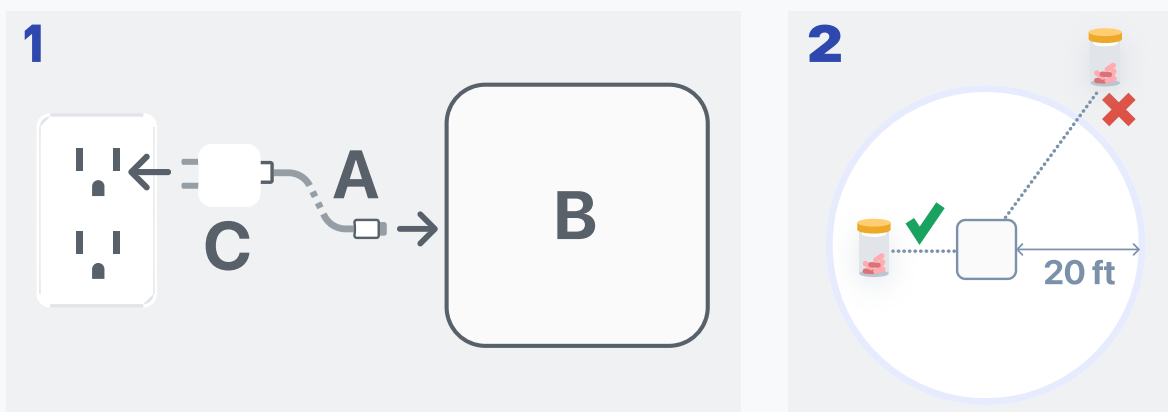

- 1** Verify that the small end of the cable (A) is plugged into the side of the gateway (B), and that the large end of the cable (A) is plugged into the wall adapter (C).
- 2** In the same room as where you store your medication, plug the wall plug (C) into an outlet.

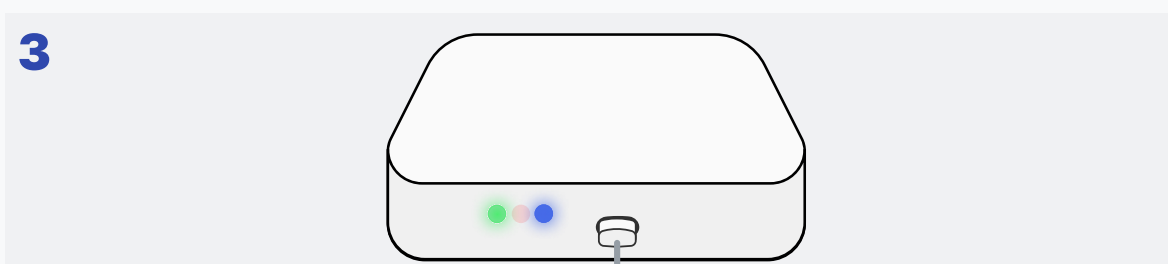

- 3** After a few minutes of blinking, the **blue** and **green** lights should become **constant**, indicating that the device is connected.

*The **red** light will blink while the device turns on. If it ever becomes **constant**, reboot the device by unplugging & replugging it back in.*

### Your Gateway setup is now complete!

If you've already set up your PatchCap, your system is now ready. Take your medications as prescribed, and we'll take care of the rest!