

tenovi

1

## Power Your Gateway


Plug your Gateway into a power outlet with the provided power cord and wait until the LED on the Gateway turns **RED**.



2

## Insert Batteries

Insert batteries into the back of the thermometer. Press the button on the front to turn the device on.

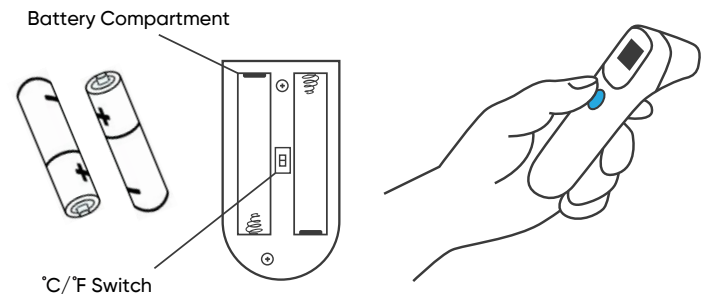

## Take A Measurement

3

Wait for the Gateway LED to spin yellow.

**Point** the thermometer at the center of the forehead. **Press the button** to take a reading.


To take in-ear measurements, consult your care provider



## Cleaning

5

- Clean forehead before use
- Clean the thermometer after use
- Wait 5-10 minutes after cleaning device to use it

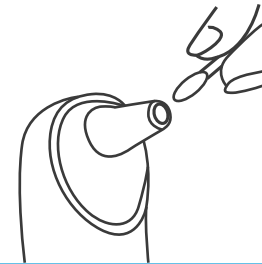

## Daily Measurements

4

Take one measurement each day unless advised by your healthcare provider. The LED light will turn **RED** at midnight then turn **GREEN** and BEEP once your first measurement for the day has been successfully transmitted. **You can continue to take additional measurements and they will be transmitted to the Gateway, but the color will not change.**



Measurement Needed



Measurement Successful



For more information about your device, use your phone to scan the QR code below or visit our support site at [tenovi.com/11](https://tenovi.com/11)

# tenovi

© 2025 Tenovi Co.  
All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Tenovi. All trademarks are owned by or licensed to Tenovi, Co. unless otherwise noted.  
Revision 11.17.25