Plug your Gateway into a power outlet with the provided power cord and wait until the LED on the Gateway turns RED.

For further assistance, please visit Tenovi.com/support.

tenoví

© 2025 Tenovi Co.

All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Tenovi. All trademarks are owned by or licensed to Tenovi, Co. unless otherwise noted.

Revision 10.22.2025

Remove Pull Tab

Remove pull tab from the battery compartment to power on the device.





5

Insert Finger Into Pulse Ox

Insert one finger completely into the measurement section of the Oximeter with your **fingernail pointing up** towards the screen.



Wait 20 seconds


Keep your finger in the oximeter for 20 seconds or until you hear a beep from the Tenovi Gateway. Then, remove your finger and wait for Tenovi Gateway to turn GREEN.





Press Power On Pulse OX

Stand within 20 feet of the Tenovi Gateway and **press the power button** on the oximeter.

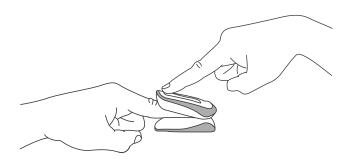

Daily Measurements

Take one measurement each day unless advised otherwise by your healthcare provider. The LED will turn RED at midnight and then turn GREEN and BEEP once your first measurement for the day has been successfully transmitted. You can continue to take additional measurements and they will be transmitted to the Gateway, but the color will not change.





Measurement Needed

Measurement Successful

6