Power Your Gateway Plug your Gateway into a power outlet with the provided power cord and wait until the LED on the Gateway turns RED.


For further assistance, please visit Tenovi.com/support.

tenoví

© 2025 Tenovi Co.


All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Tenovi. All trademarks are owned by or licensed to Tenovi, Co. unless otherwise noted.

Revision 05.01.2024

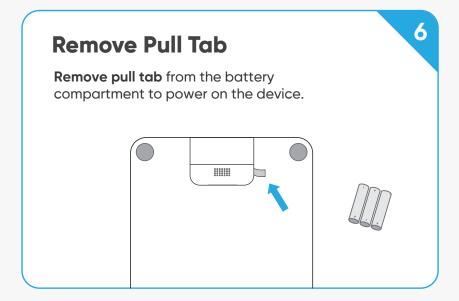


4

Scale Placement

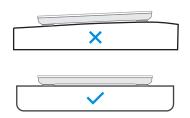

- Place your scale on a flat and hard surface.
- Do NOT place the scale on carpet or soft surface.
- Do NOT lean or hold on to anything.

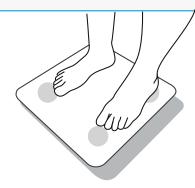

- Step on the scale with clean, bare feet.
- Wear as little clothing as possible.
- Keep your arms at your side.

Scale Operation

Do not step off the scale until the screen turns off. If the LED ring is GREEN, you have properly taken a measurement for the day and that measurement has been successfully transmitted.





Daily Measurements

Take one measurement each day unless advised otherwise by your healthcare provider. The LED will turn RED at midnight and then turn GREEN and BEEP once your first measurement for the day has been successfully transmitted. You can continue to take additional measurements and they will be transmitted to the Gateway, but the color will not change



Measurement Needed



Measurement Successful

9

5