



For more information about your device, use your phone to scan the QR code below or visit our support site at [tenovi.com/04](https://tenovi.com/04)

tenovi

© 2025 Tenovi Co.  
All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Tenovi. All trademarks are owned by or licensed to Tenovi, Co. unless otherwise noted.  
Revision 10.21.2025

1

## Power Your Gateway


Plug your Gateway into a power outlet with the provided power cord and wait until the LED on the Gateway turns **RED**.




2

## Remove Pull Tab

Remove pull tab from the battery compartment to power on the device.

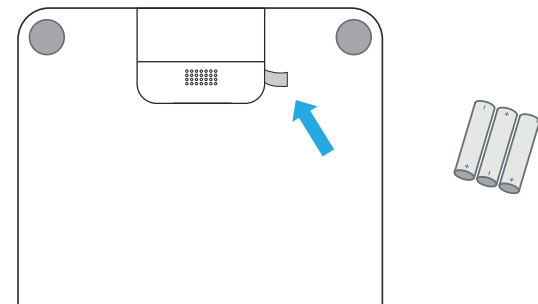

### Scale Placement

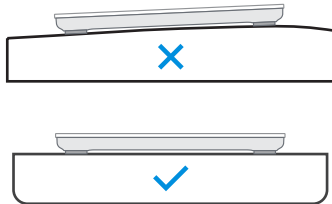

- Place your scale on a flat and hard surface.
- Do NOT place the scale on carpet or soft surface.
- Do NOT lean or hold on to anything.



- Step on the scale with clean, bare feet.
- Wear as little clothing as possible.
- Keep your arms at your side.

### Scale Operation

**Step onto the scale with bare and dry feet** and wait for the scale to power on and take measurement. When the measurement concludes, the results are shown on display. If the LED ring is **GREEN**, you have properly taken a measurement for the day and that measurement has been successfully transmitted.



### Daily Measurements

Take one measurement each day unless advised otherwise by your healthcare provider. The LED will turn **RED** at midnight and then turn **GREEN** and BEEP once your first measurement for the day has been successfully transmitted. You can continue to take additional measurements and they will be transmitted to the Gateway, but the color will not change.



Measurement Needed



Measurement Successful