

For further assistance, please
visit [Tenovi.com/support](https://tenovi.com/support).

tenovi

© 2025 Tenovi Co.
All rights reserved. Reproduction, duplication or modification
in whole or in part is prohibited without prior written permission
from Tenovi. All trademarks are owned by or licensed to
Tenovi, Co. unless otherwise noted.
Revision 05.01.2024

Power Your Gateway

1


Plug your Gateway into an outlet with the provided power cord and wait until LED on the Gateway turns **red**.



Power Your Pillbox


2

Remove the paper pull tab from the battery compartment.

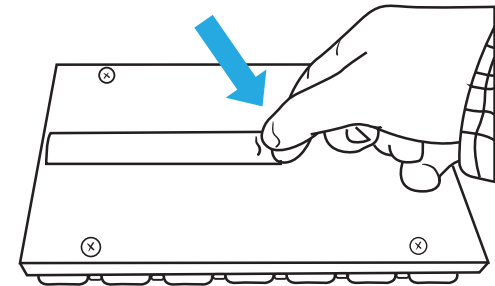


3 Fill your Pillbox

When filling your Pillbox medication, press the **refill button** at the top of the Pillbox — the compartments will start glowing. Open each compartment by pressing down on the **compartment button**.

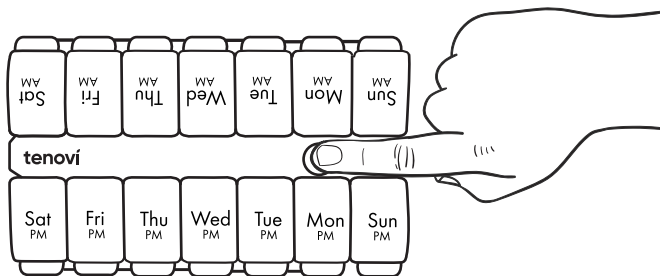


4 Exit Refill Mode

Press down on the **refill button** again when finished. Otherwise it will automatically exit refill mode after 20mins. You must use this refill button whenever you refill your Pillbox with medication.

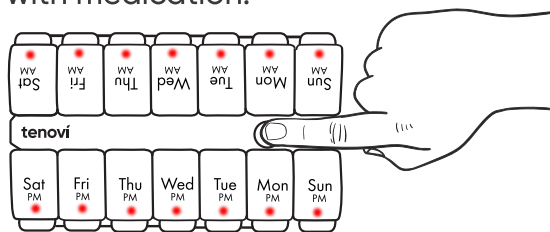

5 Take Your Medication

Take your medication as advised by your doctor. The compartment corresponding to the current day and time will illuminate when you open any compartment.

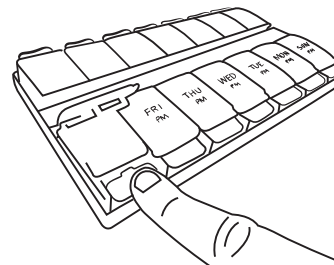

6 Use Your Pillbox Daily

Your Pillbox will store open events. When you open a compartment while near the Gateway, the Gateway LED will turn **green** and all stored events will be sent to your care provider.



Measurement Needed



Measurement Successful