

For further assistance, please  
visit [Tenovi.com/support](https://tenovi.com/support).

tenovi

© 2025 Tenovi Co.  
All rights reserved. Reproduction, duplication or modification  
in whole or in part is prohibited without prior written permission  
from Tenovi. All trademarks are owned by or licensed to  
Tenovi, Co. unless otherwise noted.  
Revision 05.01.2024

**1**


## Power Your Gateway

Plug your Gateway into a power outlet with the provided power cord and wait until the LED on the Gateway turns **RED**.


**2**

## Remove Pull Tab

Remove pull tab from the battery compartment to power on the device.

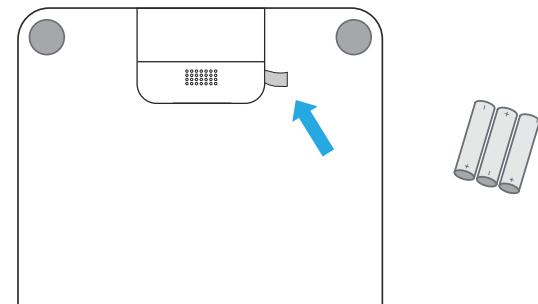

### Scale Placement

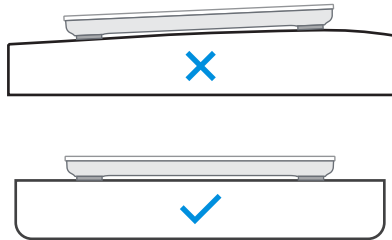

- A) Place your scale on a flat and hard surface.
- B) Do NOT place the scale on carpet or soft surface.

3



Step on the scale with clean, bare feet.  
Wear as little clothing as possible.

### Scale Operation

4

Step onto the scale with bare and dry feet and wait for the scale to power on and take measurement. When the measurement concludes, the results are shown on display. If the LED ring is **GREEN**, you have properly taken a measurement for the day and that measurement has been successfully transmitted.



### Daily Measurements

5

**Take one measurement** each day unless advised otherwise by your healthcare provider. The LED will turn **RED** each morning and then turn **GREEN** once a measurement has been successfully transmitted.



Measurement Needed



Measurement Successful