

For further assistance on how to
use your Tenovi device,
please contact a Tenovi representative
at:

Phone: (714) 418-5658

**Email:
support@tenovi.com**




© 2021 Tenovi Co.


All rights reserved. Republication, duplication or modification
in whole or in part is prohibited without prior written permission
from Tenovi. All trademarks are owned by or licensed to
Tenovi, Co. unless otherwise noted.

Revision 05.01.2021

- 1** Plug your gateway into a power outlet with the provided power cord and wait until the LED on the gateway turns red.

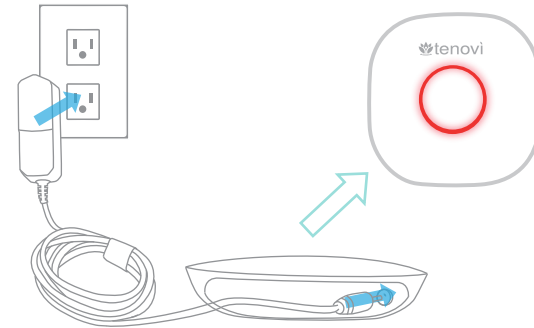


- 2** Open the battery cover and insert two AAA batteries into the back of your pulse oximeter.

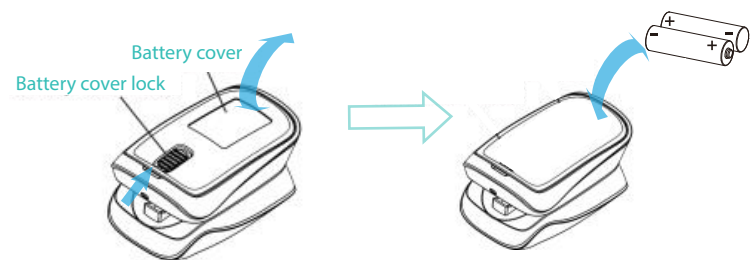


3

Insert one finger completely into the measurement section of the Oximeter with your fingernail pointing up towards the screen.

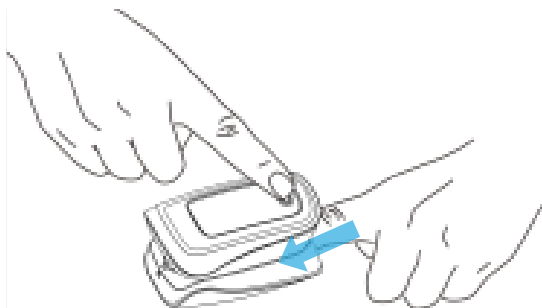


4

Stand within 20 feet of the Tenovi Gateway and press the power button on the Oximeter.

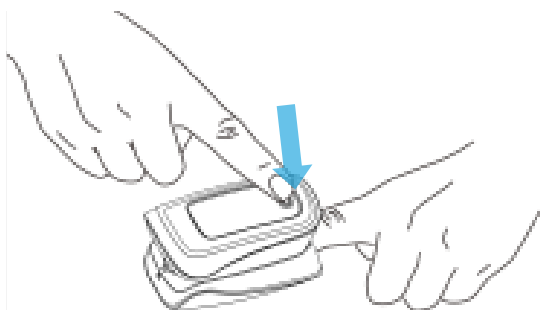

5

Keep your finger in the oximeter for 20 seconds or until you hear a beep from the Tenovi Gateway.

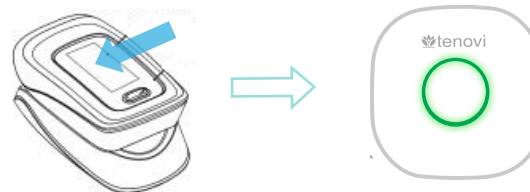

6

Take one measurement per day unless otherwise advised by your healthcare provider. The LED on Tenovi device will turn **Red** at the beginning of each day indicating you must take measurement for that day. The LED ring will turn **Green** when you successfully take a measurement and transmit it to your healthcare provider.



Measurement Needed



Measurement Successful