

Nonin Pulse Oximeter

Quick Start Guide

For further assistance on how to use your Tenovi device, please contact a Tenovi representative

Email: support@tenovi.com




© 2024 Tenovi Co.
All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Tenovi. All trademarks are owned by or licensed to Tenovi, Co. unless otherwise noted.
Revision 05.01.2024

1

Power your Gateway


Plug your gateway into a power outlet with the provided power cord and wait until the LED on the gateway turns red.

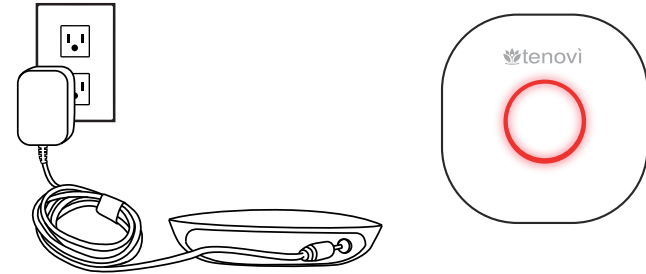

2

Insert batteries


Slide off back cover using your thumbs and insert two AAA batteries into the back of your pulse oximeter.

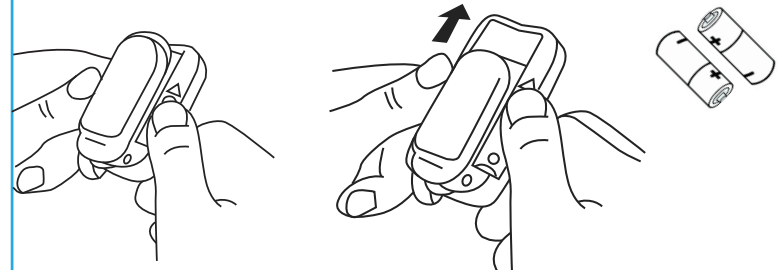

3

Insert your finger

Insert **one finger** completely into the measurement section of the pulse oximeter with your fingernail pointing down, away from the screen.

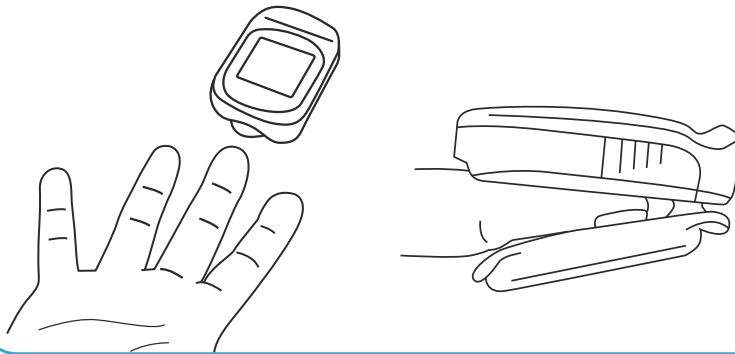

4

Take your measurement


Stand **within 20 feet** of the Tenovi Gateway. The pulse oximeter will automatically turn on and begin taking a measurement.



5

Wait for results

Keep your finger in the Oximeter for 20 seconds or until you hear a beep from the Tenovi Gateway.

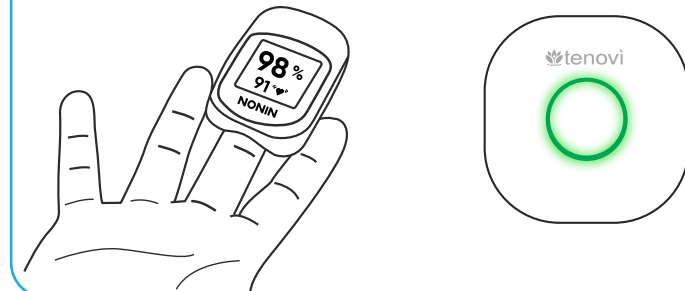

6

Daily measurements

Take **one measurement** each day unless advised otherwise by your healthcare provider. The LED will turn **red** each morning and then turn **green** once a measurement has been successfully transmitted.



Measurement Needed



Measurement Successful