


Tenovi Cellular Pulse Oximeter

Quick Start Guide

3

Press Button


Press the **power button** on the oximeter. Keep your finger in the oximeter for 20 seconds

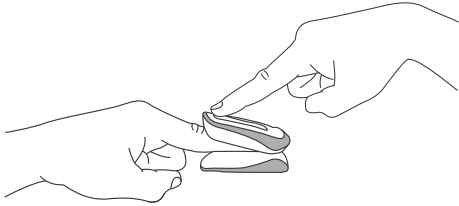

4

Wait 20 seconds

Once the measurement has been completed, the device will then lock the reading and begin to transmit the data over the cell network.

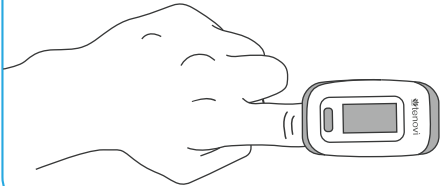

5

Measurement Taken

"**Success**" will display when the reading has been transmitted. When "**FAILED**" is shown, the reading will be saved and the device will try and send again on your next measurement.

Success

For further assistance on how to use your Tenovi device, please contact a Tenovi representative

Email: support@tenovi.com



Tenovi Cellular Pulse Oximeter

Quick Start Guide




© 2024 Tenovi Co.
All rights reserved. Reproduction, duplication or modification
in whole or in part is prohibited without prior written permission
from Tenovi. All trademarks are owned by or licensed to
Tenovi, Co. unless otherwise noted.
Revision 05.01.2024

1

Remove Pull Tab

Remove **pull tab** from the battery compartment to power on the device.

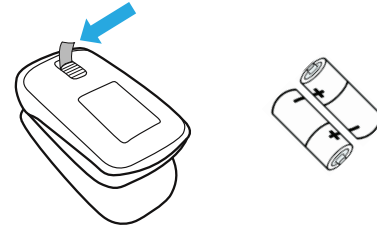

2

Insert Finger Into Pulse Ox

Insert one finger completely into the measurement section of the Oximeter with your fingernail **pointing up** towards the screen.

